

⚠ Contact the technical services department


Calls

Patient calls 	Call from the door unit in room 1.
	Call from bed 1 in room 1.
	Call from bed 2 in room 1.
	Call from the pull cord in the bathroom in room 1.

Biomedical alarm in room 1 	Secure wandering alarm on door 1
---------------------------------------	---





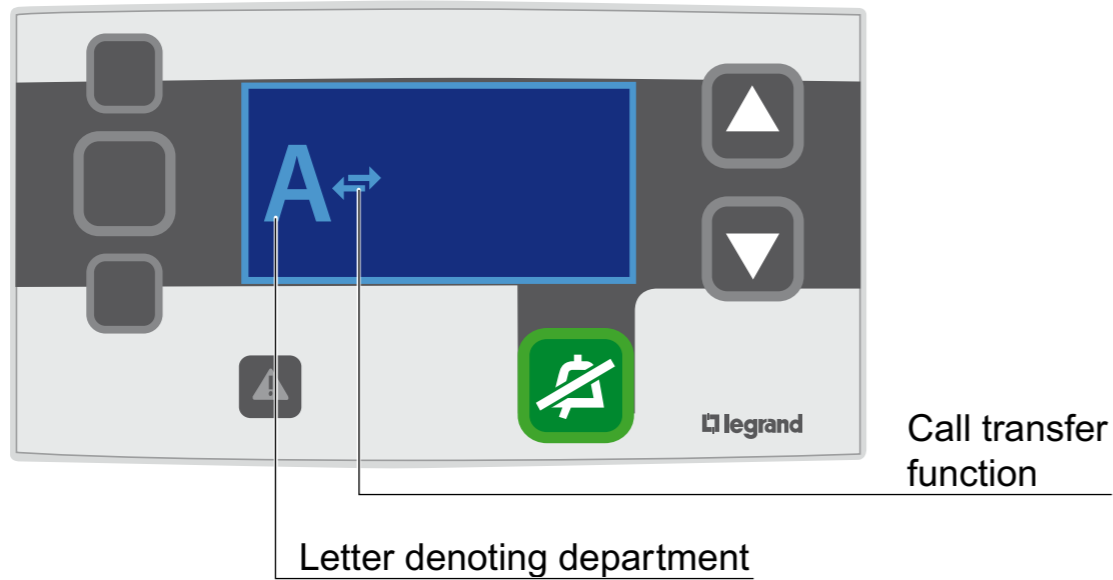


Indications on the display	
	Nurse present in room 1.
	Attendance indicated by pressing the button on the main control unit (sound muted).

Nurse assistance request in room 1 	Nurse present Nurse assistance request
Blue code alarm in room 1 	Nurse present High-level emergency request
Bathroom emergency in room 1 	Nurse present Request for assistance
Blue code alarm in the bathroom in room 1 	Nurse present High-level emergency request








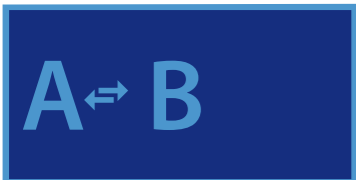

List of departments	
Letter	Department
<input type="text"/>
<input type="text"/>
<input type="text"/>
<input type="text"/>

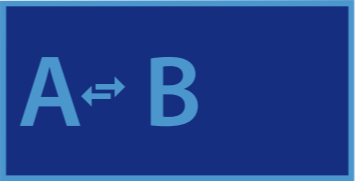To transfer a call to another department

Main control unit with call transfer function enabled 	Press the  button on the control unit.
Configuring the call transfer on the control unit 	Select the department to which you wish to transfer the call by pressing the  button
Confirming the transfer 	Confirm the transfer by pressing the  button on the control unit.
Transferring call from dept. A to B 	

To cancel the call transfer

Transferring call from dept. A to B 	
Confirming the transfer 	Press the  button on the control unit.
Configuring the call transfer on the control unit 	Select the "empty" department to cancel the transfer by pressing the  button.
Main control unit with call transfer function enabled 	Press the  button on the control unit.